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## Face Surgical Mask Type II Technical Datasheet

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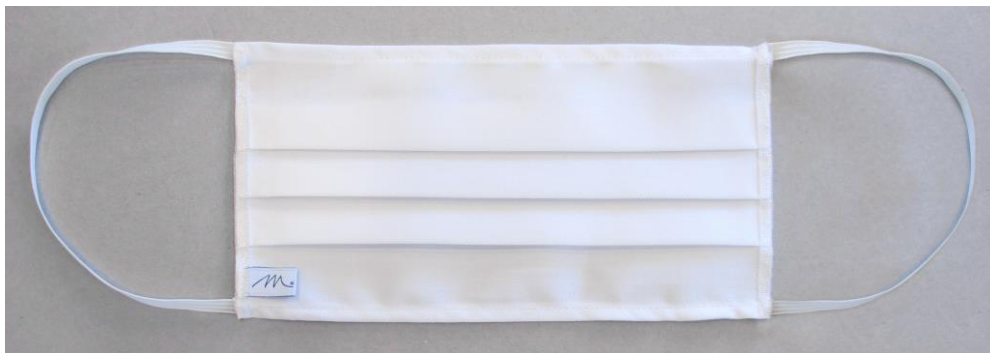
REGISTRATION ID 1961169

Art. 15 of Italian Legislative Decree No. 18 of 17/03/2020

OMOLOGAZIONE ISTITUTO SUPERIORE SANITA' PROTOCOLLO 17657/COV-DM-D42 del 19/05/2020 /  
 APPROVED BY ISS (ITALIAN NATIONAL INSTITUTE OF HEALTH) PROTOCOL 17657/COV-DM-D42 19/05/2020

**MASCHERINA CHIRURGICA LAVABILE 20 VOLTE – PRODOTTA IN ITALIA**  
**WASHABLE SURGICAL MASK 20 TIMES – MADE IN ITALY**

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Dimensioni chiusa / Dimensions when closed (mm):

Regular 185 x 90	Small 185 x 90	XS 175 x 90	Junior 160 x 75
------------------	----------------	-------------	-----------------

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**Indications as to whether or not the product is sterile and any sterilisation methods**

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Test effettuati presso Laboratorio / Tests carried out at the following Laboratory:

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**Tipo II e IIR** (a seconda se resistenti o meno agli schizzi) destinate agli operatori sanitari / **Type II and IIR** (depending on whether or not they are splash-resistant) which are intended for healthcare professionals

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## ENGLISH

### 5 Contraindications and/or Warnings - Directions of use

#### Warnings

*"The WHO recommends only using a face mask if you suspect you have contracted the Novel Coronavirus and are experiencing symptoms such as coughing and sneezing, or if you are taking care of a person with a suspected Novel Coronavirus infection", states the Italian Ministry of Health website. "The face mask must be used in combination with other hand hygiene and respiratory measures, as these remain the most essential measures for reducing the possibility of infection."*

Therefore, the rules for proper use are indicated below:

- Before putting on a mask, clean your hands with an alcohol-based hand sanitiser or soap and water.
- When covering your mouth and nose, make sure there are no gaps between your face and the mask.
- Avoid touching the mask while using it and, if necessary to do so, clean your hands with an alcohol-based hand sanitiser or soap and water beforehand.
- Replace the mask with a new one as soon as it becomes damp or sanitise it according to the procedures indicated in Par. 8.
- To remove the mask: remove it from behind (without touching the front).
- Immediately discard the mask in a closed bin; then clean your hands with an alcohol-based hand sanitiser or soap and water.
- Should the mask require sanitising, follow the rules indicated in Par. 8.
- Furthermore, please note that there are no proven benefits to wearing multiple overlapping masks.
- Keep a distance of 1 metre between you and other people.

### 6 Indications as to whether or not the masks are single-use or washable

As a result of the type of materials used to produce the face mask, it can be rewashed at least 20 times as tested in accordance with UNI EN ISO 4920, providing that the product does not show any signs of minimal wear. Carefully follow the face mask sanitisation procedures found in Par. 8.

### 7 Face mask packaging

Face mask production and packaging cycle:

1. Piece laying
2. Piece cutting
3. Pleating
4. Edge sewing
5. Elastic attachment by side sewing
6. Ozone sanitising
7. Packaging
8. Labelling and traceability

Production procedures

1. Layer cutting: performed by a section cutter.
2. Pleating: performed using a special guide and ironed with a continuous press
3. Edge seam: carried out using a professional linear sewing machine
4. Side seam: carried out using a professional overlock machine
5. Elastic attachment: carried out using a professional linear sewing machine
6. Sanitisation by ozone cabinet
7. Packaging: performed manually using polypropylene bags
8. Labelling and traceability: a batch number is assigned, and a label is affixed on each package.

### 8 Procedures for a proper sanitisation

#### Proper sanitisation information

See: [Enteric Virus Survival during Household Laundering and Impact of Disinfection with Sodium Hypochlorite](#)

*The most important factors for reducing the virus in fabrics were found to be tumble drying the load after washing and the addition of sodium hypochlorite. Washing with washing powder alone was not found to be effective at removing or inactivating enteric viruses, as significant concentrations of virus were found on the samples.*

*The use of sodium hypochlorite (bleach) reduced the number of infectious viruses on the samples after washing and drying by at least 99.99%.*

#### Sanitising at High Temperatures

1. We recommend washing the product in the washing machine at a temperature of at least 60°C with a bleach-based additive and a subsequent dryer cycle at the maximum available temperature. If you do not own a dryer, line dry the product. If the washing machine does not have a specific compartment for bleach, you can place it directly in the drum; before adding the bleach, start the washing cycle first so that the drum fills up with water. Never put the face masks in the washing machine with undiluted bleach. Read the instructions on the bleach's packaging to find out the correct dose to use, according to the size of the load.

#### Sanitising at Cold Temperatures:

1. We recommend using a hydroalcoholic solution (70% alcohol) or, more simply, a similar disinfectant, such as Bialcol. Apply the product on both sides of the face mask, making sure to go over the laces and elastics again. Take care to not wet the mask excessively in order to avoid facilitating the deterioration of the materials. Leave for approximately **30 minutes**.
2. Cold water and bleach can also be mixed. To prepare a disinfectant soaking solution, dilute the bleach in cold water; the exact dosage depends on how much fabric you need to wash. Dilute 15 ml of bleach for every 4 litres of cold water up to a maximum of 20 litres. Make sure the fabrics have already been washed before soaking them with bleach. Leave for approximately **15 minutes**. Rinse the washing in very hot water and then wash it the washing machine. Once it has been immersed in the solution for the appropriate time, rinse it thoroughly with very hot water; next, put it in the washing machine on a normal wash programme in order to remove any traces of bleach.
3. Wash or soak the fabrics in hydrogen peroxide and Borax. Prepare a solution to pour into the washing machine or soak the clothes. If you opt to pour the solution into the washing machine, mix 1 litre of hydrogen peroxide with 400 g of Borax, in addition to your usual washing powder; only add the products once the washing machine has filled with water.  
If you choose to soak the fabrics instead, mix 1 litre of hydrogen peroxide with 400 g of Borax in a container that has already been half-filled with water. Leave the fabrics in the solution for **15-30 minutes**. Rinse with very hot water and then wash in the washing machine as normal at a very high temperature.

## 9 Disposal methods

The product must be discarded in general waste bins

## 10 Quality management system

The face masks are produced at our factory, in compliance with precise procedures that guarantee:

- The traceability of all components used.
- Checks on the incoming materials to be used.
- Checks along the entire production chain.
- The traceability and labelling of every single package.

## 11 Contacts

For information and orders write to [mascherine@monti.it](mailto:mascherine@monti.it) - [www.monti.it](http://www.monti.it)